

Cleaning Autorefractor Lens

(Save for autorefractor and DO NOT USE ON EYEGLASSES)

NOTICE

Check the lens for cleanliness regularly ESPECIALLY before you open your clinic.

Kit has cleaning tissue, Q-Tips and lens cleaner.




Spray on end of Q-Tip



With Q-tip, gently wipe clean right, middle and left lens on autorefractor.

Right

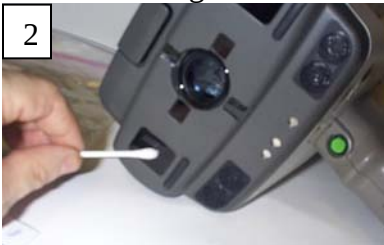

Middle



Left



Using cleaning tissue, **gently** wipe dry right, middle & left lens on autorefractor.

Right



Middle



Left



NOTICE

After wiping in steps 5-7, look at the lens in the light at an angle to be certain there are NO smears. If there are, then using a new tissue and Q-Tip, repeat steps 1-7 above until the lens are clean.

Return cleaning kit to autorefractor case.

