

👖 Kendall Optometry Ministries, Inc 👖



Operating the Reichert LensChek Lensmeter

By **Holland Kendall**

5/4/2006



"Serving the Lord by providing better vision to the people of underdeveloped countries." Phone: (502)-640-2227 E-Mail: hollandkendall@peoplepc.com Web: http://eyeglasses-inventory.com

Table of Contents

1.0	Introduction	3
2.0	Setting Up Glasses Reader program	
3.0	Single Vision Eyeglasses.	
4.0	BiFocal Eyeglasses	8
5.0	Progressive Eyeglasses	11
6.0	In Summary	

1.0 Introduction

The purpose of this document is to show how to use the Reichert LensChek digital lensmeter to measure different types of eyeglasses and to print barcoded labels (of the prescription) to go onto the eyeglasses bags. These barcoded eyeglasses will then go to Christian mission teams going all over the world.

To help you to identify which kind of these eyeglasses, refer to the KOM document entitled "How Glasses Change Your Vision". It is a good idea for you to study this document if you plan to measure eyeglasses. By understanding the concepts described in this document, you will better understand how to identify the type of eyeglasses that you are measuring.

There are 4 basic types of eyeglasses.

- 1) Single Vision Eyeglasses. These eyeglasses only have one area of magnification. Typically they are used just for one purpose: distance, intermediate, or reading.
- 2) Bifocal Eyeglasses. These eyeglasses have a dual purpose in mind. They are used for both distance and reading.
- 3) Trifocal Eyeglasses. Sometimes there is a third area of magnification on the eyeglasses sandwiched between the bottom and top areas of magnification. If this is present, then the eyeglasses are called trifocals.
- 4) Progressive lens eyeglasses. Progressive lens eyeglasses have mostly replaced bifocals and almost entirely replaced trifocals. Progressive lens eyeglasses have their least power in the center for distance and progressively get more powerful as you move down to the bottom edge of the eyeglasses.

The following sections describe the measurement of these 4 types of eyeglasses. Bifocals and trifocals will be described together in the same section.

Keep in mind that all 4 of these types of eyeglasses can have an additional parameter called "prism". Prism is typically used to correct a problem with double vision. When a large amount of prism is cut into used prescription eyeglasses, they will not likely be usable by another person. This type of eyeglasses should be discarded.

At this point use the document entitled "How Glasses Change Your Vision" to identify the pair of eyeglasses you are planning to measure. Place your eyeglasses in the instrument as shown below and then go to the appropriate section which follows.



Be certain the bottom edge of the frame stays positioned on the platform as shown.

Setting Up Glasses Reader program. 2.0

Start the Glasses Reader program (if not already) by clicking on the icon on the desktop which looks like you see to the right.

Status

<u>C</u>lear Record FLAGS? Sunglasses? Reading Glas

<u>Save &</u> Clear

You should then see the screen to the right. This indicates that the program is running and ready to accept measurements.

Be sure to enter your initials in the circled block so that they will be printed on the eyeglasses that you measure.

If you do not see this initial screen (a different inventory number is OK) then click on **Options** and set the screen to be as you see to the right. (Circled settings are critical) The COMM Port number may be different for your computer. Make any necessary changes and click Save to File and Continue. Keep in mind the name of your printer might also be different than what you see. If you have a Dymo Labelwriter, select it. The initial program screen should then be right.

This program also requires barcode fonts to print barcode labels. You will need therefore to purchase Elfring Barcode Fonts:

http://www.elfring.com/bar39.htm

Go to far right hand side and click "BUY NOW 1". You can download for \$99 or order a CD for \$9.50 more.

The purpose of this program is to produce barcoded labels like you see to the right. These go on the eyeglasses bag and are used by the teams for rapid entry of eyeglasses into their inventory.



kender

3.0 Single Vision Eyeglasses.

Single vision eyeglasses only have one area of magnification. Typically they are used just for one purpose: distance or reading but not both. Keep in mind that you can easily be deceived in believing you have a pair of single vision eyeglasses when actually you have a pair of progressive vision eyeglasses. Be sure you have single vision eyeglasses before you proceed. Check the "Progressive Identification Chart" in Appendix B of the "How Glasses Change Your Vision" document.

Here are the steps you will perform.

- 1. Place your eyeglasses into the instrument as shown in section 1.
- 2. Setup the instrument so that it shows the following screen.





Push here to select the symbol showing to the left of the button.

You do this by pressing the button pointed to above (right) until the display shows the above setting.

Keep in mind the picture above this button displays three different states as shown below.



Pressing the button causes the LensChek to switch to the next mode. The "Contact Lens" choice will not be used.

3. Now move the eyeglasses up and down (using the platform levers on the left or right) and to the left or the right until the bulls-eye target appears in the center of the screen as shown in the below sequence. Make certain that the frame is always placed snugly on the platform. Use the arrows to determine which way you should move the eyeglasses on the platform.



-0525 - 100 177

When the eyeglasses are moved into the center you will see the picture on the right above. The right lens has now been measured.



4. Place the left lens into the instrument. (below left)

0525 - 100 176

5. Move it around and position it by raising the platform and moving the lens to the right and left until the picture shows as indicated in the below center. Press the memorize button

Left lens in instrument.

Move to get bulls eye

Press "memorize" button



Press "Print" button.







6. Press the lensmeter PRINT button as shown above to transfer the reading to the Glasses Reader program. You should see a response something like the below.



7. Click either "For male, For Female, Either Male or Female, or Child" and then click the PRINT button circled above. The label will be printed. Put the label on the eyeglasses bag and place it in the box. Make sure the label is turned sideways (from end to end) on the eyeglasses bag.

4.0 **BiFocal Eyeglasses**

BiFocal eyeglasses have two areas of magnification. Typically they are used for distance and reading. The top area is for distance while the bottom area is for reading. Keep in mind that also there are TriFocal eyeglasses which have a bottom, middle and top segment. When you have one of these pairs of eyeglasses, you should measure it like bifocals and measure just the top and the bottom segments and ignore the middle segment.

Here are the steps you will perform.

- 1. Place your eyeglasses into the instrument as shown in Section 1.
- 2. Setup the instrument so that it shows the following screen and follow the indicated steps in the picture.



3. Now move the eyeglasses up and down (using the platform levers on the left or right) and to the left or the right until the bulls eye appears in the center of the screen as shown in the below sequence. Make certain that the frame is always placed snugly on the platform.

Move up to right.

Stay above BiFocal line.

Bulls eye.



4. Now measure the add portion of the lens using the below sequence.

Press to select "add". Center lens in Bifocal.

Press "memorize"



Reading is completed.



- 5. Place the left lens into the instrument. (Below picture on the left)
- 6. Select the left lens and move it around and position it by raising the platform and moving the lens to the right and left until the picture shows as indicated below.



7. Switch instrument to "add" and move the lens down until you get a reading on the lower part of the lens (the add). Press the "memorize" button.

Press the "add". In BiFocal press "memorize" Press "print" to send to PC.

8. Press the lensmeter PRINT button to transfer the reading to the Glasses Reader program. You should see a response something like the below:

Glasses Reader	r Data Collection Progra	MTT.					Version 3.	1-3 📃 🗖
Status:	USING: File Output Disa NO ERRORS.	Reich	ert LensC	hek Le	nson	neter	ts options	Help IClick here PRINT Label to
		L	ensmeter Help (Chc	k here] <-L	ast Emp	TOO TO	moly-2	
	Left (OS)			Ri	ght (O	D)		DYM0 LabelWriter 330-U58
	Sphere +0.75	Both		5	obere	+1.00	Both	successfully
	Cylinder: -1.75	Both		Cy	linder:	50	Both	
/	Axis: 00	Both			Axis:	80	Both	
Jear Record	Add: +2.50	Both			Add:	+2.25	Both	Recall Last Entry
FLAGS? Sunglasses Beading Gla	? Progressive? ssses? Ø Bi-Tritocal?	Scratched?	For Male?	Either Ma or Female For Child	ale Fi	rame Size Small? Medium?	T Large	PD mm
Save &								Exit
2 Processed	successfully	Commen	t on Glasses (Use NO co	ommas	Enter ye	our Initial	e.
Reichert Lens	Chek Right Sph: + 1.00	Cyt50 X	80# ADD: + 2.25 L	eft Sph: + 0.3	75 Cyl: - 1	1.75 × 00#	ADD: + 2.5	01
		Curren	t Instrument R	eading				
Reichert Lens	Chek Right Sph: + 2.25	Cyt00 X	00ø Left Sph: + 2.	25 Cyt50	× 03# 1			
Data Inpu	It Selection:	Prior	Instrument Re	ading (2 1			
Barcode	Data Entry?					Check for	Records Fig	tre Holo

9. Click either "For male, For Female, Either Male or Female, or Child" and then click the PRINT button circled above. The label will be printed. Put the label on the eyeglasses bag and place it in the 50 pair box. Make sure the label is turned sideways (from end to end) on the eyeglasses bag.

VERY IMPORTANT

When measuring bifocal lens, avoid positioning the top measurement so that the light beam of the LensChek goes through the Bifocal segment line (top part of the bifocal segment). This will cause a very high erroneous cylinder reading.

If you have this problem and cannot get a reading without moving into this position, move the eyeglasses down with the beam above the line and force a reading by pressing the grey "memorize" button.

5.0 Progressive Eyeglasses

Progressive lens eyeglasses have mostly replaced bifocals and almost entirely replaced trifocals. Progressive lens eyeglasses have their least power in the center for distance and progressively get more powerful as you move down to the bottom edge of the eyeglasses. When you measure these pairs of eyeglasses, you must be very careful as to your selection of the top portion of the lens. Position the eyeglasses high up on the lens until the instrument tells you to move to a lower position.

Here are the steps you will perform.

- 1. Place your eyeglasses into the instrument as shown in Section 1.
- 2. Select the progressive mode by setting the instrument as below (See red arrow). Select the mode. You should see the below.



3. Select the right side and place the right lens in the instrument as shown below.



Adjust until you see a bowtie.

Bowtie.



3. Set lensmeter to "add" and then move the lens to view the bottom part of the lens. You may have to move straight down the lens or in the direction of the nose.



Move eyeglasses up to near the bottom of the lens.



Move eyeglasses until you see.

Set to "add" position.

THIS pattern.

Press the "memorize"



4. Move the eyeglasses to measure the left lens as shown below.

Position to left side.

Move eyeglasses.

Start adjusting position.



Must move to the left.

Moving lens to the left.

Got a bow tie.



5. Move the eyeglasses down near the edge to measure the bottom part (the add) and then select the "add". Gradually move down until you get the highest "add" value displaying (on the screen area circled below) but stop before touching the frame.

Select "add". Position until you se the below Memorize the measurement.

To record the bottom portion of the eyeglasses remember that you must push the circled grey "memorize" button as shown in the above right. Pressing PRINT (shown on the left above) transfers the reading to the computer.

6. The screen of the PC will now show the below.

	USING	Poid	had I and a		Res	ports Option	is Hanko
Status:	File Output Disa	hled	nert LensChel	k Lens	ometer		Parter Holy
	NO ERRORS.		A A	8		-	PRINT
			4	10000 1	000 100	븮 무-	Label to:
			Lenzmetes Help [Lick here	C-Last	Emply Next	Empty-3	DYMO
	Left (OS)			Right	(00)		LabelWiter 330-USB
				100	(00)		3 printed
	Sphere +0.25	Both		Sphe	te +0.00	Both	successfully.
	Cylinder: -1.75	Both		Cylind	ler: -1.00	Both	
	Axis: 108	Both		A	as 79	Both	
	Add: +1.75	Both			11-1-00	Dath	
Clear Record					11.00	Dota	Recall Last
FLAGS?					Frome Size	7	
Reading Et	7 Progressive? asses? Di-Trilocal?	Scratched	7 For Male 7 9 10 1 For Female 7 1 Fo	Female a Child?	Small? St Median?	E Lorge	mm
Save &							Evit
Clear 3 Processed	successfully	Commer	nt on Glasses (Use)	NO comm	as) Fators		
							_
Beichert Lens	Chek Prog Add Lens B	aht Sale -	0.00 Cet - 1.00 X 73+ ADI	0 - 1 00 Lei	1 Sph + 0.25	Cyt - 1.75 X	108# ADD:
		Curren	t Instrument Readin	HQ.			
Reichert Lens	Chek* Right Sph + 1.00	Cyt - Sex	80a ADD + 2.25 Left Sph	+ 0.75 Cel	-1.75 X 80# /	00 + 2.50	
		Prior	Instrument Reading	(3)			1
Data Inpu	It Selection -						
E Barcada	Data Entry?						
Lancone	cond cany?				Chick for B	lancardes Loda	y Help

Click either "For male, For Female, Either Male or Female, or Child" and then click the PRINT button circled above. The label will be printed. Put the label on the

eyeglasses bag and place it in the box. Make sure the label is turned sideways (from end to end) on the eyeglasses bag.

6.0 In Summary

Hopefully this document has helped you to understand how to use the Reichert LensChek lensmeter in combination with the Kendall Optometry Ministry, Inc Glasses Reader program.